


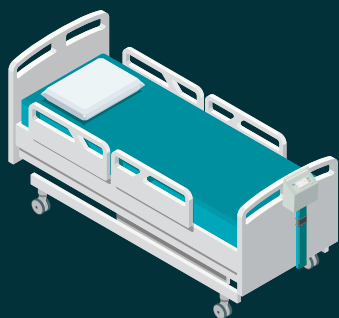

Step by step

Before starting the mattress, the patient should be **positioned in supine** (face up, centered in bed).

Use positioning cushions, if needed. Ensure that heels are properly offloaded.



To start the turning program, press start.

The mattress will begin curving. This process will take approx. 35 mins, and then the continuous turning starts.

For patient care mode (static), press the start button once more while the program is running. This activates a 30 min. stop.




To stop the turning program, press stop.


Standard settings gives 20° turning degree both ways, at medium turning speed and a 15 min delay on each side.



See our instruction video

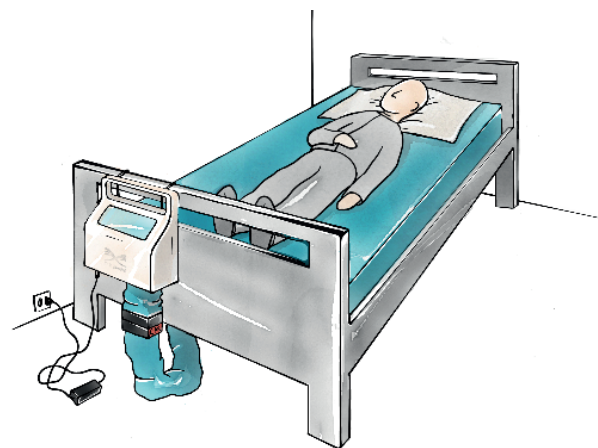

ON - Light on
CARE MODE - Light blinking
OFF - Light off

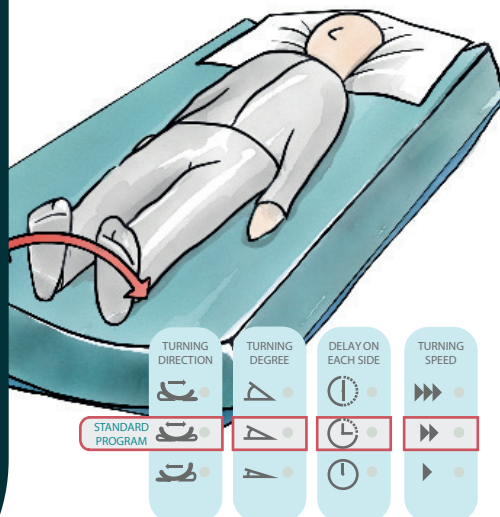

Choose settings

Turning Direction

It is possible to exclude the patient's sides from being loaded.



Exclude the patient's right side

Turn in both directions

Exclude the patient's left side

Turning degree

Adapt the turning degree to the patient's health condition.



Patient is turned to 30°

Patient is turned to 20°

Patient is turned to 10°

Delay on each side

The turning can be delayed at the end of a turn based on need.



30 minutes delay on each side

15 minutes delay on each side

Continuous movement, no delay

Turning speed

Choose between 20–60 minutes from one side to the other.



High speed: 20 minutes

Medium speed: 40 minutes

Low speed: 60 minutes




Custom settings

To choose a setting, press the round button accompanying the individual setting, one to three times. The active setting lights up on the panel.



See our instruction video

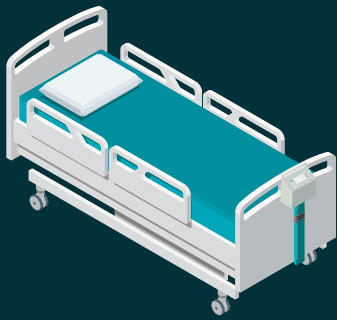

Step by step

Place the mattress in bed with the hose towards the foot end. With the foam side pointing up.

Guide the hose through the base of the bed. Make sure it is not squeezed or pinned.




Attach the pump unit to the foot end of the bed and connect it to the mattress (audiable click).


Connect the power cord to the pump unit and then to a wall socket. Then wait 30 seconds.

After 30 seconds the panel on the pump unit will light up. The mattress is now ready for use.



See our
instruction video

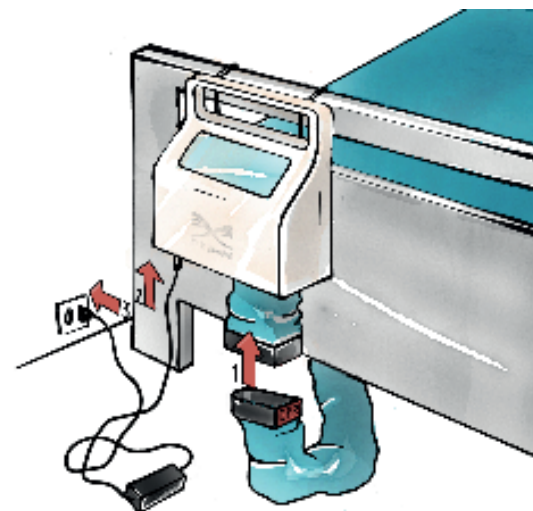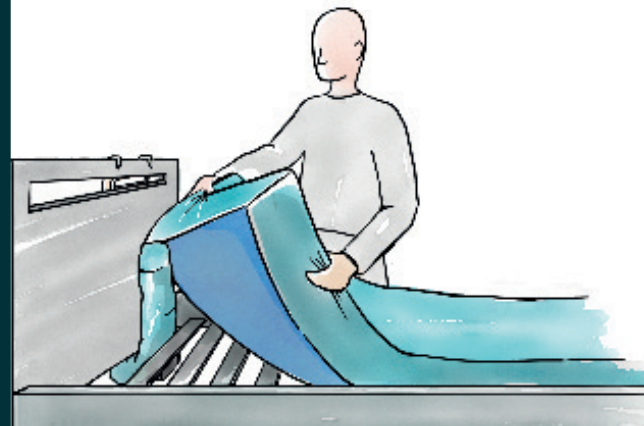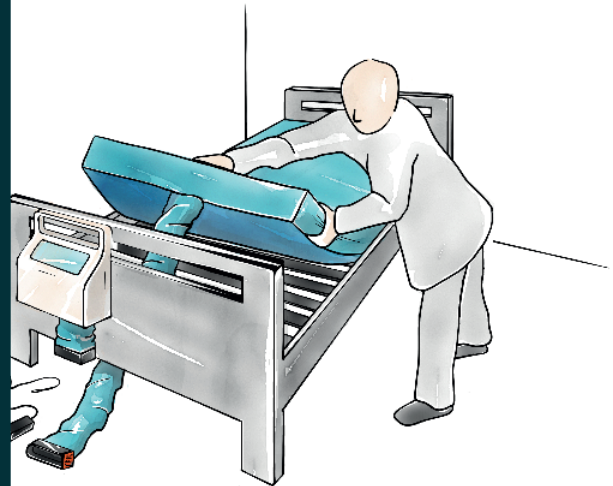

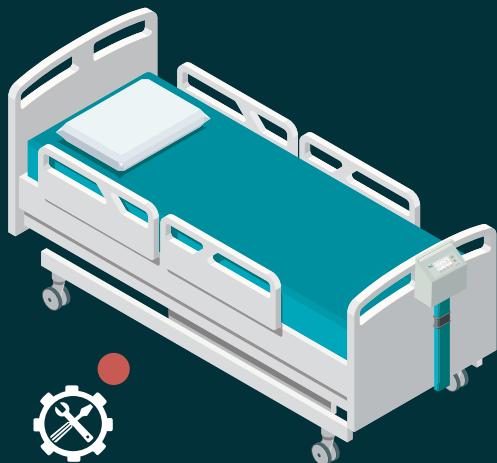

Check connector

Check if the air hose is connected, pull it gently to make sure the connector is properly engaged.

Check cable

Make sure the air hose is not in a tight bend or pinched between the mattress and bed frame.



More than 30° backrest

If the backrest is raised above 30°, the turning program will stop and the >30° lights up.




When the backrest is lowered to 30° or below, the mattress will automatically resume.

Check the user manual

If none of the above seem to be the problem, check out the troubleshooting section of the user manual or call your service technician.

QUICK GUIDE TROUBLESHOOTING

Tidewave™ Mattress

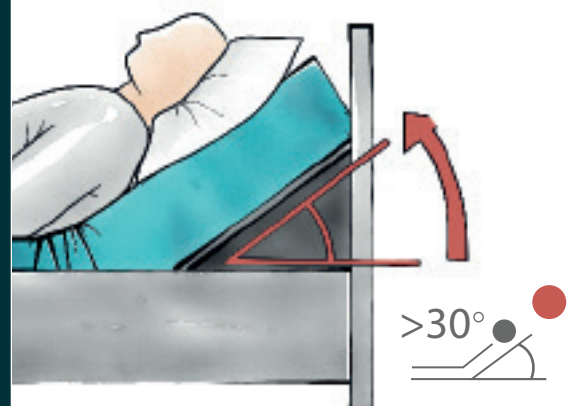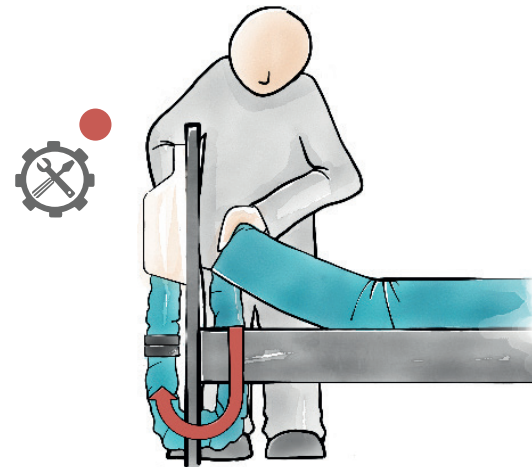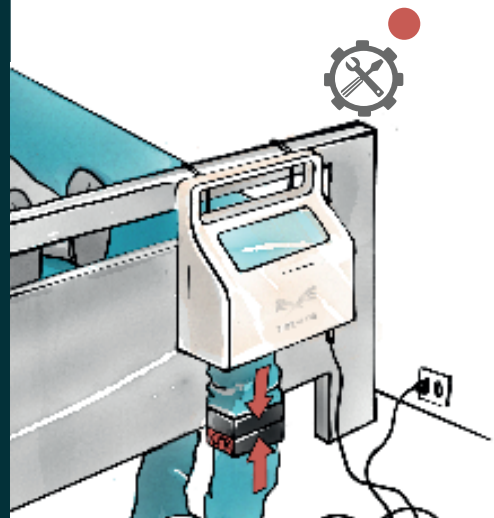

See our user manual